



Stago Canada Ltd/Stago Canada Ltée  
251 Consumers Rd., Suite 1200  
Toronto, Ontario (Canada)  
M2J 3R4  
Phone: 1-888-283-0175  
info@stago.com  
www.stago.com

February 18, 2011

Re: Creating a saving disk on the STA Compact® Analyzer

Dear Valued Customer:

Please follow the below procedure for creating a saving disk on your STA Compact®. If this procedure is not followed correctly, it will not allow you to successfully create a saving disk. This saving process should be performed as part of the monthly maintenance or when any test settings have been changed. Please refer to the Maintenance section in the STA Compact® Reference Manual for more information.

**Creating a saving disk:**

- Select **Maintenance** from the Main Menu
- In the User Maintenance Menu, select **Saving**
- With **Selection of Items** highlighted, press **Enter**
- Press the space bar to select or deselect an item from the list
  - Select **Test Configurations** and **System parameters**
  - **\*\*\*DO NOT select Error Messages. This will cause unsuccessful saving.**
  - **\*\*\*DO NOT select F8 to select all items**
- Once these 2 items are selected, press **F10** to confirm selection
- Insert a **pre-formatted floppy disk** into the floppy drive located in the front left panel
- Highlight **Save** and press **Enter** three (3) times to confirm
- If an error occurs during saving, please try a **NEW pre-formatted floppy disk**
- Once saving is complete, label the disk with serial number of instrument and date of saving

Please contact the Hotline at 1-888-283-0175 with any questions.

Sincerely,  
*Alicia Carpenter*  
Technical Training Manager

